
## Large Blood Pressure Cuff

For all CVS or Microlife upper arm blood pressure monitors.



Thank you for purchasing a CVS accessory cuff. This cuff is suitable for arm circumferences of 12" to 16". The cuff is easy to adapt to all units.

For automatic monitors simply remove the existing cuff from the monitor and plug the large cuff into the monitor. (Skip to section A)

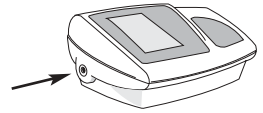



**For semi-automatic (manual inflation) monitors follow these steps:**

1. Remove the T connector from the unit.
2. Detach the hose that leads to the cuff.
3. Remove the connector on the end of the new cuff.
4. Attach replacement cuff's hose to the T connector. Then connect the T connector to the monitor.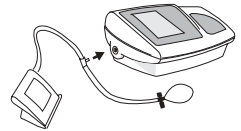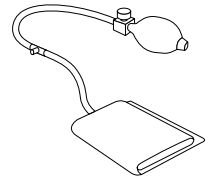

**A - Once the cuff is properly connected to the monitor, prepare the cuff for your measurement.**

1. Thread cuff end through metal loop and slide cuff onto left arm. The cuff is properly placed when the bottom edge is just above the elbow crease, and the "artery mark" is directly over the brachial artery.
2. Fasten the Velcro just tight enough so you can slide two fingers under the cuff.
3. Sit comfortably and rest arm at heart level with palm up.

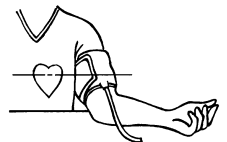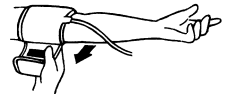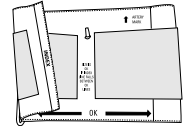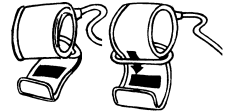**Be sure to consult the instruction booklet included with your monitor for correct use of your blood pressure monitor.**

**Questions? Call the Customer Help Line Toll Free at 1-866-464-6184.**

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